ACTI-BONE®

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ACTI-BONE®

HYALURONIC ACID
STERILE SOLUTION
(for dental use).

ENG

Indications

Implant surgery:

- a) as an adjuvant treatment in the regeneration of bone defects of the peri-implant alveolar ridge in association with bone substitutes and resorbable membranes.
b) protection of the implant surface, implant site and sutures.

Treatment of peri-implantitis:

as an adjuvant treatment in bone regeneration processes also in association with bone substitutes and with resorbable membranes.

Extractive surgery:

as an adjuvant treatment in the bone regeneration of the alveolus and prevention of post-extraction alveolitis.

Instructions for use

Aspirate the desired amount of solution from the vial with the sterile syringe.

Implant surgery:

a) as an adjuvant treatment for the regeneration of peri-implant bone defects, mix the product with the bone particles until a homogeneous mixture is obtained, remove the excess solution with a sterile gauze, then apply the product in the bone defect modeling it with a special sterile instrument.

b) to protect the implant and the implant site, apply the film-forming solution bending needle to mimic a periodontal probe. Place the needle tip near the base of the implant site and inject the product until the solution comes out of the upper margin. Pull the needle out of the site. Apply few drops of the product on the implant immediately before insertion into the site and make sure that the entire surface is wet.

Treatment of peri-implantitis:

mix the product with the bone particles until a homogeneous mixture is obtained, eliminate the excess solution with a sterile gauze, then apply the product in the bone defect modeling it with a special sterile instrument.

Extractive surgery

(in particular for delayed implant placement):

mix the product with the bone particulate until a homogeneous mixture is obtained, eliminate the excess solution with a sterile gauze, then apply the product in the alveolus modeling it with a special sterile instrument.

Dispose the syringe after use.

Packaging Contents

- n° 1 multi-dose bottle 5 ml sterile solution.
n° 5 sterile disposable syringes 2,5 ml (Artsana CE 0373).

Composition

Sodium hyaluronate, monobasic potassium phosphate, dibasic sodium phosphate, water for injections.

Warnings

- Medical Device for dentist use only,

- Sterile product,
- Multi-dose bottle for at least 5 applications,
- Do not reuse the needles and syringe: throw them immediately after use,
- Not for injection,
- Keep away from heat sources and out of reach of children,
- Do not use after the expiration date,
- Store at a temperature not exceeding 25 ° C.
- The unused solution, if stored in the fridge at 4° temperature, maintains the efficacy for 5 weeks.

Legend of symbols

	LOT NUMBER
	MEDICAL DEVICE
	EXPIRATION DATE
	STERILIZATION BY HEAT
	STORAGE TEMPERATURE +25°C
	KEEP AWAY FROM DIRECT SUNLIGHT
	CAREFULLY READ THE INSTRUCTIONS BEFORE USING

Italmed S.r.l.
Viale Mazzini, 15
50132 Firenze, Italia

CE 0373

Rev. 00 - 02/2023